detax

Ohrabformsilikone

addition ultra®

addition supra®

addition spectra

addition rapid

addition mini Junior

addition Junior DIC

detax flextime®






pink mix

addition protect

de	Gebrauchsanweisung	4
en	Instructions for use	10
fr	Mode d'emploi	14
es	Modo de empleo	18
it	Istruzioni per l'uso	22
da	Brugsanvisning	26
el	Οδηγίες χρήσης	30
hr	Upute za uporabu	34
lv	Lietošanas instrukcija	38
lt	Naudojimo instrukcija	42
nl	Gebruiksaanwijzing	46
no	Brugsanvisning	50
pl	Instrukcja stosowania	54

pt	Manual de instruções	58
ro	Instrucţiuni de utilizare	62
sv	Bruksanvisning	66
sk	Návod na použitie	70
sl	Navodila za uporabo	74
cs	Návod k použití	78
hu	Használati útmutató	82
ru	Инструкция по применению	86
tr	Kullanım Talimatları	90
et	Kõrvajäljendi silikoon	94
fi	Korvajäljennössilikoni	98
bg	Силикон за снемане на отпечатък на ухо	102

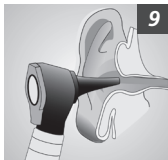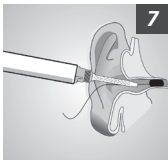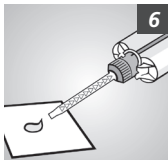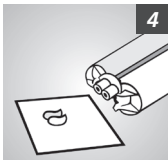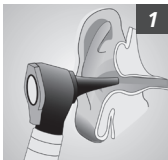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Ohrabformsilikon

INDIKATION

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Ear impression silicone

INDICATION

Manufacture of an impression of the external auditory canal for determination of shape and dimension in the production of earmolds.

PATIENT TARGET GROUP

Persons for whom an earmold is to be created.

INTENDED USERS

ENT specialist / hearing aid acoustician

PRODUCT DESCRIPTION

Silicone-based elastomeric precision impression materials, addition-curing, for impression-taking of the external auditory canal.

PROCEDURE

PREPARATION OF THE EAR

Before taking the impression, carefully examine the auditory canal and the eardrum (Fig. 1). In case of any pathological findings (e.g. injuries, inflammation or perforation of the eardrum, etc.), a specialist must be consulted before an impression may be taken. The ear canal must be cleaned and depilated. Insufficient cleaning may delay or inhibit the curing process due to the presence of cerumen. It is mandatory to protect the eardrum with an impression pad before taking an impression. Place the impression pad at the end of the outer ear canal in front of the eardrum (Fig. 2). The impression silicone can then be applied directly into the pretreated, tamponed ear, whereby care is to be taken to avoid bubbles.

MIXING AND METERING

The cartridges are emptied with the Automix pistol (Fig. 3). After inserting the cartridge into the mixing gun, remove the cartridge closure by turning it. Extrude a small amount of material until silicone is being uniformly emitted from both outlet openings (Fig. 4). To attach the mixing cannula, pay attention to the guides on the cannula and cartridge. Lock by turning clockwise (Fig. 5). Emit material with even pressure. Spread a small amount of material before use and visually check that the base and catalyst pastes are homogeneously mixed (Fig. 6). Only then can the metering be carried out individually. Slowly apply the impression material into the ear canal and the concha (Fig. 7). During the spraying process, make sure that the tip of the cannula remains at the level of the injected material in order to avoid air inclusions. After curing, the impression can be carefully removed and further processed (Fig. 8). Finally, the ear must be checked for intactness (Fig. 9). After use, leave the mixing cannula as a closure cap on the cartridge until the next application.

SAFETY INFORMATION

- ▶ Only for the specified intended use by trained specialists.
- ▶ Do not bring into contact with condensation-curing silicones.
- ▶ Cured impression materials are chemically resistant – avoid stains on clothing.
- ▶ In case of stiffness or clogging discard the cartridge; do not use force for application.
- ▶ Do not leave any residues of the impression material in the ear.
- ▶ Latex gloves and latex-contaminated surfaces, cerumen, creams and plastic materials can inhibit curing (we recommend nitrile or polyethylene gloves).
- ▶ The processing instructions and precautionary measures must be observed. Otherwise, irreparable damage to the hearing organ or eardrum may result.
- ▶ After each impression taking, careful examination of the auditory canal is necessary.
- ▶ Wear suitable personal protective equipment during the finishing of the cured material.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

NOTES

- ▶ Detax shall not be held liable for any damage caused by misuse.
- ▶ Read and understand the safety data sheet!

For users and/or patients:

Any serious incidents occurring in relation to this product should be reported immediately to incident@detax.com and to the competent authority of the Member State in which the user and/or patient is established.

CONTRAINDICATION

Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Silicone impression materials have been tried and tested millions of times; undesired effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

DISPOSAL

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

Technical Specifications

	Product color [base/catalyst]	Mixing volume	Automix system	Consistency		Time in the ear Min. approx.
				low	high	
addition ultra®	blue/white	50 ml cartridge	Automix 1 & 2	▼.....		3:00
addition supra®	pink/white	50 ml cartridge	Automix 1 & 2	▼		3:00
addition spectra	lilac/white	50 ml cartridge	Automix 1 & 2	▼		2:30
addition rapid	blue/white → white	50 ml cartridge	Automix 1 & 2	▼		1:30
addition mini Junior	yellow/white	10 ml cartridge	minimix	▼.....		2:00
addition Junior DIC	yellow/white	50 ml cartridge	Automix 2	▼.....		2:00
detax flextime®	purple/lilac	50 ml cartridge	Automix 1	▼		2:30
pink mix	pink/white	50 ml cartridge	Automix 2	▼		2:30
addition protect	turquoise/light blue	50 ml cartridge	Automix 2	▼		1:55

Processing:

at 23 °C ± 2 °C / 73 °F ± 4 °F,
50 ± 5 % rel. humidity

Storage:

25 °C
77 °F
15 °C
59 °F

* from start of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F,
50 ± 5 % rel. humidity. Higher temperatures shorten,
lower temperatures extend the indicated times.

Total setting time Min. approx.*	Final hardness	Mixing cannulas		Particularities
		Automix1	Automix2	
3:30	37 Shore A	 optima	 pink	universal
3:30	35 Shore A	 optima	 pink	narrow auditory canals, ITE / CIC
3:00	39 Shore A	 optima	 pink	low extrusion force
2:00	30 Shore A	 optima	 pink	Color change during curing, extra short time in the ear
2:30	25 Shore A		 minimix green	Impression taking for children & adolescents
2:30	25 Shore A		 yellow	Special impression material for DIC adaptations
3:00	35 Shore A	 optima		thermocontrolled
3:00	38 Shore A		 green	intermediate consistency, high sealing
2:25	38 Shore A		 pink	Ear protection impression, intermediate sealing

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detax GmbH