# detax

## Ohrabformsilikone

pink putty

detax addition

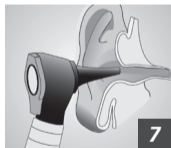
green eco

detax flextime®

addition futur

<b>DE</b>	Gebrauchsanweisung .....	4	<b>NO</b>	Bruksanvisning .....	37
<b>EN</b>	Instructions for use .....	7	<b>PL</b>	Instrukcja stosowania .....	40
<b>FR</b>	Mode d'emploi .....	10	<b>PT</b>	Manual de instruções.....	43
<b>ES</b>	Modo de empleo .....	13	<b>RO</b>	Instrucțiuni de utilizare .....	46
<b>IT</b>	Istruzioni per l'uso .....	16	<b>SV</b>	Bruksanvisning .....	49
<b>DA</b>	Brugsanvisning.....	19	<b>SK</b>	Návod na použitie .....	52
<b>EL</b>	Οδηγίες χρήσης.....	22	<b>SL</b>	Navodila za uporabo .....	55
<b>HR</b>	Upute za uporabu .....	25	<b>CS</b>	Návod k použití .....	58
<b>LV</b>	Lietošanas instrukcija.....	28	<b>HU</b>	Használati útmutató.....	61
<b>LT</b>	Naudojimo instrukcija .....	31	<b>RU</b>	Инструкция по применению .....	64
<b>NL</b>	Gebruiksaanwijzing .....	34	<b>TR</b>	Kullanım Talimatları .....	67

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Ohrabformsilikon

## INDIKATION

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Ear impression silicone

## **INDICATION**

Manufacture of an impression of the external auditory canal for determination of shape and dimension in the production of earmolds.

## **PATIENT TARGET GROUP**

Persons for whom an earmold is to be created.

## **INTENDED USERS**

ENT specialist / hearing aid acoustician

## **PRODUCT DESCRIPTION**

Silicone-based elastomeric precision impression materials, addition-curing for impression-taking of the external auditory canal.

## **PROCEDURE**

### **PREPARATION OF THE EAR**

Before taking the impression, carefully examine the auditory canal and the eardrum (Fig. 1). In case of any pathological findings (e.g. injuries, inflammation or perforation of the eardrum, etc.), a specialist must be consulted before an impression may be taken. The ear canal must be cleaned and depilated. Insufficient cleaning may delay or inhibit the curing process due to the presence of cerumen. It is mandatory to protect the eardrum with an impression pad before taking an impression. Place the impression pad at the end of the outer ear canal in front of the eardrum (Fig. 2). The impression silicone can then be applied directly into the pretreated, tamponed ear, whereby care is to be taken to avoid bubbles.

### **METERING AND MIXING**

Withdraw components A (catalyst) and B (base) using the appropriate metering spoons (Fig. 3) and mix rapidly at a ratio of 1:1 until a uniform color is achieved (Fig. 4). Fill the homogeneous mass into an impression syringe, apply slowly into the auditory canal and the concha (Fig. 5), taking care to avoid bubbles. During the injection process, make sure that the tip of the impression syringe remains at the level of the injected material. After curing, the impression can be carefully removed and further processed (Fig. 6). Finally, the ear must be checked for intactness (Fig. 7).

► See figures on page 3

## **SAFETY INFORMATION**

- ▶ Only for the specified intended use by trained specialists.
- ▶ The processing instructions and precautionary measures must be observed. Otherwise, irreparable damage to the hearing organ or eardrum may result.
- ▶ Make sure that only base and catalyst components from the same batch number are used.
- ▶ Do not bring into contact with condensation-curing silicones.
- ▶ Cured silicon materials are chemically resistant – avoid stains on clothing.
- ▶ Do not bring components A + B into contact before mixing.
- ▶ Do not mix up measuring spoon or lids.
- ▶ Do not leave any residues of the impression material in the ear.
- ▶ Latex gloves and latex-contaminated surfaces, cerumen, creams and plastic materials can inhibit curing (we recommend nitrile or polyethylene gloves).
- ▶ After each impression taking, careful examination of the auditory canal is necessary.
- ▶ Wear suitable personal protective equipment during the finishing of the cured material.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

## **NOTES**

- ▶ Detax does not accept liability for any damage caused by misuse of the impression material.
- ▶ Read and understand the safety data sheet!

### **For users and/or patients:**

Any serious incidents occurring in relation to this product should be reported immediately to [incident@detax.com](mailto:incident@detax.com) and to the competent authority of the Member State in which the user and/or patient is established.

## **CONTRAINDICATION**

Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Silicone impression materials have been tried and tested millions of times; undesired effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

## **DISPOSAL**

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

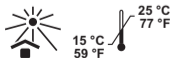
## Technical Specifications

	pink putty	detax addition	green eco	detax flextime®	addition futur
<b>Product color</b> [Base/catalyst]	pink/white	blue/white	green/yellow	purple/pink	purple/ light blue
<b>Mixing volume</b>	2 × 500 ml 2 × 1000 ml	2 × 250 ml 2 × 670 ml 2 × 5000 ml	2 × 800 g 2 × 25 kg	2 × 250 ml	2 × 250 ml
<b>Texture, consistency</b> low high					
<b>Open time</b> Min. approx.*	1:30	2:00	2:00	2:00	1:30
<b>Indwelling time in the ear</b> Min. approx.	2:30	3:00	3:00	2:00	3:30
<b>Total setting time</b> Min. approx.*	4:00	5:00	5:00	4:00	5:00
<b>Final hardness</b>	35 Shore A	30 Shore A	40 Shore A	30 Shore A	25 Shore A

### Processing:

At 23 °C ± 2 °C, 50 ± 5%  
rel. humidity

### Storage:

### \* from start of mixing at

23 °C ± 2 °C, 50 ± 5% rel. humidity.  
Higher temperatures shorten,  
lower temperatures extend the  
indicated times.

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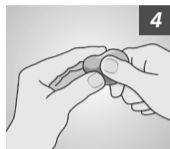
### **pink putty**

Standard packing 03510

base + catalyst 2 x 500 ml  
2 measuring scoops

Eco packing 03511

base + catalyst 2 x 1000 ml  
2 measuring scoops

### **detax addition**

Standard packing 02381

base + catalyst 2 x 250 ml  
2 measuring scoops

Large packing 02717

base + catalyst 2 x 670 ml  
2 measuring scoops

Mega packing 02271

base + catalyst 2 x 5000 ml  
2 measuring scoops

### **green eco**

Standard packing: 02487

base + catalyst 2 x 800 ml  
2 measuring scoops

Bulk packing: 02703

base + catalyst 2 x 25 kg

### **detax flextime®**

Standard packing 02300

base + catalyst 2 x 250 ml  
2 measuring scoops

### **addition futur**

Standard packing 04019

base + catalyst 2 x 250 ml  
2 measuring scoops

### **easy**

Standard impression syringe

blue 02140

Spare cylinder

transparent 02492

### **easy2**

CIC impression syringe  
long, pink 03503

Spare cylinder  
long, transparent 03189

impression plugs secure  
conical 03321

# detax

Caution: Federal U.S. law restricts this device to sale by or on the order of a licensed healthcare practitioner (or trained specialist personnel).

CE 0483

**detax GmbH**

Carl-Zeiss-Str. 4

76275 Ettlingen, Germany

+49 7243 510 0

[detax.com](http://detax.com)

03/2026

6