

Annex 1

Digital light Processing (DLP) printer, operation software and parameter

Model (Picture)	Printer Manufacturer, Model	Light source	Light intensity	Operation Software	Parameter data set*
	Asiga Max	385 nm	6.1 mW/cm ²	Composer 1.2.11	Detax_Freeprint model 2.0 385 COLOR_5
	Asiga Max 2	385 nm	7.0 mW/cm ²	Composer 2.0.8	Detax_Freeprint model 2.0 385 COLOR_5
	Asiga Pico 2	385 nm	20 mW/cm ²	Composer 1.2.11	Detax_Freeprint model 2.0 385 COLOR_5
	Asiga PRO 2	385 nm	5.7 mW/cm ²	Composer 1.2.11	Detax_Freeprint model 2.0 385 COLOR_5
	Asiga PRO 4K	385 nm	7.0 mW/cm ²	Composer 1.2.11	Detax_Freeprint model 2.0 385 COLOR_5
	Asiga Ultra	385 nm	6.6 mW/cm ²	Composer 2.0.8	Detax_Freeprint model 2.0 385 COLOR_5

*The set of parameters includes all relevant material- and printer specific information
The placeholder **COLOR** shall be replaced by **caramel, grey, light grey, sand** or **white**

Annex 1

Digital light Processing (DLP) printer, operation software and parameter

Model (Picture)	Printer Manufacturer, Model	Light source	Light intensity	Operation Software	Parameter data set*
	Microlay Eve Pro	405 nm	3.5 mW/cm ²	Microform 1.0.3.7	DETAX Freeprint Model 2.0 385 100 microns v2.1
	Microlay Versus	385 nm	4.3 mW/cm ²	Microform 1.0.3.7	DETAX Freeprint Model 2.0 385 COLOR 100 microns v5.3
	Miicraft Alpha	385 nm	6.0 mW/cm ²	Utility 6.4.4	DETAX Freeprint model 2.0
	Miicraft Prime Series	385 nm	6.2 mW/cm ²	Utility 6.3.0	Detax_Freeprint model 2.0 385_100_PH
	Miicraft Ultra Series	385 nm	5.7 mW/cm ²	Utility 6.3.0	Detax_Freeprint model 2.0 UV_100
	Phrozen Sonic 4K 2022	405 nm	2.0 mW/cm ²	Phrozen DS Slicer	DETAX Freeprint model 2.0

*The set of parameters includes all relevant material- and printer specific information
The placeholder **COLOR** shall be replaced by **caramel, grey, light grey, sand** or **white**

Annex 1

Digital light Processing (DLP) printer, operation software and parameter

Model (Picture)	Printer Manufacturer, Model	Light source	Light intensity	Operation Software	Parameter data set*
	RapidShape D20+/ D30+/ D40+	385 nm	2.0 mW/cm ²	Netfabb 2020	DETAX Freeprint model 2.0 385 COLOR
	Shining 3D AccuFab D1s	405 nm	7.6 mW/cm ²	AccuWare	DETAX FREEPRINT model 2.0 385
	Shining 3D AccuFab L4D	405 nm	2.2 mW/cm ²	AccuWare	DETAX FREEPRINT model 2.0 385
	Way2Produc- tion SoliFlex Series	385 nm	8.0 mW/cm ²	Netfabb 2020	Freeprint model 2.0 385 COLOR

*The set of parameters includes all relevant material- and printer specific information
The placeholder **COLOR** shall be replaced by **caramel**, **grey**, **light grey**, **sand** or **white**

Cleaning Equipment

Cleaning unit	
Manufacturer, Model	Cleaning process **
Rapidshape RS wash	Use the following settings: DETAX Freeprint model 2.0 385 COLOR Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.
Ultrasonic bath Bandelin Sonorex	Clean the parts with isopropyl alcohol (purity $\geq 98\%$) for 3 minutes. Then thoroughly clean the openings, cavities and gap areas with compressed air. The main cleaning is performed in a separate vessel with fresh isopropyl alcohol (purity $\geq 98\%$) for 3 minutes. Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.

The placeholder **COLOR shall be replaced by **caramel, grey, light grey, sand** or **white**



detax GmbH

Carl-Zeiss-Str. 4 • 76275 Ettlingen • Germany
+49 7243 510 0 • post@detax.com • detax.com

07/2025

Light curing Equipment

Light Curing unit	
Manufacturer, Model	Curing process***
Dentalfarm Photopol	2 x 6 min, 120 % + N ₂ , turn around components after 6 min
Formlabs FormCure	30 min
Meccatronicore BB Cure	30 min; 85 % Power
NK Optik Otoflash G171	2 x 2000 flashes under inert gas, turn around components after 2000 flashes
NK Optik Otoflash 250/500	4000 flashes under inert gas @15 Hz
Phrozen PhrozenCure	30 min
Prusa Research Medical CW One	30 min
Rapidshape RS cure	Use the following settings: DETAX Freeprint model 2.0 385 COLOR
Rapidshape RS cure XL	Use the following settings: DETAX Freeprint model 2.0 385 COLOR
Shining 3D FabCure	30 min

***The placeholder **COLOR** shall be replaced by **caramel, grey, light grey, sand or white**



detax GmbH

Carl-Zeiss-Str. 4 • 76275 Ettlingen • Germany
+49 7243 510 0 • post@detax.com • detax.com

07/2025