detax

3D Freeprint® Material

# splintmaster

---

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Resin for dental 3D printing

**Indication**

Splints, retainers, mouthguards, nightguards, bleaching trays

**Patient Target Group**

Persons being treated in the context of a dental procedure.

**Intended Users**

Dentist, dental technician

**Suitable for the following DLP printers/cleaning/post-exposure**

see "Annex 1" (enclosed separately)

**Processing**

- The properties of the final product depend, among other things, on post-processing. Correct post-exposure is important for biocompatibility. Therefore it must be ensured that the light unit is in an orderly condition and that the moulds are completely cured (refer to the process description).
- Before use, the material in the bottle should be shaken intensively and homogenized with a bottle roller.
- Minimum material thickness for the design: flex: 1,0 mm; taff 0,8 mm
- Polish surface mechanically.
- Processing temperature (at 23 °C ± 2 °C / 73 °F ± 4 °F).

**Safety Information**

- Only for the specified intended use by trained specialists.
- Avoid direct contact with the liquid material and the components before post-curing, especially in pregnant / breastfeeding women. Irritating to airways, eyes and skin (sensitization possible).
- Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- After contact with eyes rinse thoroughly with water immediately and consult a doctor.
- After contact with skin wash immediately with water and soap.
- Biocompatibility is only guaranteed with complete polymerisation.
- Refer to the relevant safety data sheet for hazard and safety information.

## Cleaning

### By dental laboratory / practice

In dental laboratories and practices, the cleaning is done in ultrasonic baths.

### By the patient

For daily cleaning, use lukewarm water, a soft toothbrush, liquid soap or detergent. Then rinse well afterwards. Cleaning tablets can be used following the manufacturer's instructions.

### Note

- Do not use toothpaste; the abrasive particles are coarse and scratch the surface.
- Do not use any acids or solvents.
- Do not use mouthwash.

## Notes

- Detax shall not be held liable for any damage caused by misuse.
- Always keep container tightly sealed, immediately close the container carefully after each use.
- Please follow the instructions on the safety data sheet!

### For users and/or patients

Any serious incidents occurring in relation to this product should be reported immediately to [incident@detax.com](mailto:incident@detax.com) and to the competent authority of the Member State in which the user and/or patient is established.

## Storage

- Store Freeprint® splintmaster flex/taff in a dry place 15 °C - 28 °C / 59 °F - 83 °F and away from light. Even slight exposure to light can trigger polymerization.
- To protect against contamination, cover the material in the tray with the lid or a glass plate.

## Contraindication

Contains (meth)acrylics and phosphine oxide.

Some ingredients of Freeprint® splintmaster flex/taff may cause allergic reactions in predisposed persons. In such cases refrain from using the product. Freeprint® splintmaster flex/taff only insert intraorally in completely polymerised state.

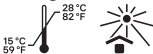
## Adverse effects

Product may cause allergic reactions.

## Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

## Storage

## Application

At 23 °C ± 2 °C / 73 °F ± 4 °F

## Manufacturing

Data preparation and fabrication of the support structure according to the instructions of the CAD software manufacturer

### ① Construction process

Generation of a Print Job complying with machine and material parameters

### ② Post-processing

After raising the platform, a drip time of approx. 10 minutes is recommended. If possible, post-processing should commence immediately following the construction process.

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**detax GmbH**