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[illegible]

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Hedef hasta grubu

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Application Aids

Patient Target Group

Persons being treated in the context of a dental procedure.

Intended Users

Dentist

Application needles

Product Description

Application needles for Laser Lock application syringes
K03056: Application needles long, black for application of smartseal® 30
K03214: Application needles curved, black for application of smartseal®

Procedure

Remove the sealing cap of the application syringe. Place the application needle on the application syringe and latch the Laser Lock by pulling it out until it is locked. Then check for secure fit. Further procedure according to the instructions of the product to be used.

Applicator

Product Description

The applicator is used for highly precise placement of liquids. The head consists of non-absorbing fibers. The solutions to be applied are held between the fibers. A non-dripping sphere is formed, which holds the liquid to be applied until the application has been performed.

Procedure

The applicator can be easily bent to highly precise application of solutions in poorly accessible areas. The head remains in the desired position without spring back. The applicator is designed for the application of smartseal® cervi. Further procedure according to the instructions of the product to be used.

Intra-oral tips / Contouring tips

Product Description

Intra-oral tips and contouring tips serve as attachment for mixing cannulas in the Automix1 or Automix2 system. They are used for targeted application of light-bodied silicone in the sulcus and preparation area.

Contouring tips are shaping and generating a strand for simplified application of bite registration silicone.

Procedure

The application aids are attached by pressing them into the mixing cannula until they snap into place. Then check for secure fit. Preparation and further procedure according to the instructions of the product to be used.

Safety Information

- Only for the specified intended use by trained specialists.
- For single use only.

Notes

- Dextas shall not be held liable for any damage caused by misuse.

For users and/or patients:

- In any serious incident that has occurred in connection with this product must be reported immediately to info@dentax.com and to the competent public authority of the country in which the user and/or patient resides.

Contraindication

Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Undesirable effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend application of bite registration for alternative.

Disposal

Disposal must be carried out in accordance with the local/regional/national and international regulations.

Intended use:
Application aids for dental use

Indications:
Application needles:
Attachments for direct application
with Laser Lock connection

Applicator: Training and application
of liquids

Intra-oral tips: For Application of
impression silicone

Storage:

Ordering information:

Application needles

Application needles,
black long 25 pcs. **02456**

Application needles,
black curved 10 pcs. **02714**

Applicator

Applicators
pink 100 pcs. **03543**

**Intra-oral tips /
Contouring tips**

Intra-oral tips

white 96 pcs. **02345**

Intra-oral tips
yellow 96 pcs. **02712**

Contouring tips
12 pcs. **02611**

Contouring tips
12 pcs. **02648**

Assignment of mixing cannulas:

Intra-oral tips white:
Mixing cannulas pink 02707
T-mixer pink 03614
Mixing cannulas 6 mm 02344

Intra-oral tips purple:
Mixing cannulas purple 02704

Intra-oral tips yellow:
Mixing cannulas yellow 02706
T-mixer yellow 03615
Mixing cannulas 10.1,
yellow 02807

Contouring tips:
Mixing cannulas 6 mm 02344

Contouring tips:
Mixing cannulas green 02705

Application needles | Applicator |
Intra-oral tips / Contouring tips

dentax

MADE IN GERMANY

11/2025

Caution: Federal U.S. law prohibits this device to be used on the order of a dentist or other health care professional

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© 2025 dentax GmbH