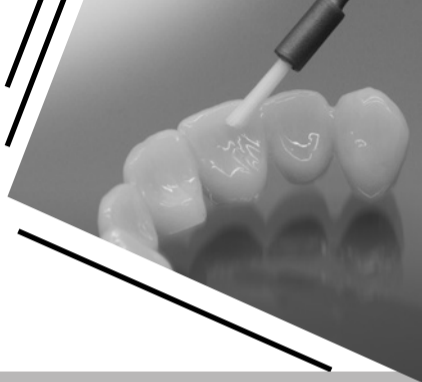
# detax

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**INTENDED USE**

**smartrepair**®

Repair material for temporary acrylic in dental prosthetics

**smartrepair**® **pink**

Modeling and repair material for temporary acrylic in dental prosthetics

**smartbond**®

Bonding for repair material

**tempofit**® **finish**

Surface sealing for temporary acrylic in dental prosthetics

**INDICATION**

**smartrepair**®

- ▶ Repair of temporary crown and bridge restorations.
- ▶ Reinforcement or completion of temporary crowns and bridges.

**smartrepair**® **pink**

DENTAL OFFICE:

- ▶ Esthetic adaptation and correction of the gingival areas of temporary crown and bridge restorations.
- ▶ Chairside repairs of acrylic-based dentures and removal of bubbles, cracks and pores from them.
- ▶ Reattachment of teeth and repairs of dentures.

LAB:

- ▶ Fracture and fissure repairs of dentures.
- ▶ Completion and contouring in the base area.
- ▶ Partial completions and relinings.
- ▶ Manufacture of small immediate single-tooth dentures.

**smartbond**®

Bonding for **smartrepair**®/**smartrepair**® **pink**.

**tempofit**® **finish**

Do not process the surface sealing of the temporary crown and bridge restorations on the patient.

**PATIENT TARGET GROUP**

Persons being treated in the context of a dental procedure.

**INTENDED USERS**

Dentist, dental technician

## PRODUCT DESCRIPTION

### smartrepair®

Light-curing acrylic-based material specially developed for repairs and completion of temporary crowns and bridges. It is available in the following colors: A2, A3.

### smartrepair® pink

Light-curing acrylic-based modeling and repair material specially developed for esthetic adaptation of the gingival area.

### smartbond®

Light-curing bonders for smartrepair®/smartrepair® pink.

### tempofit® finish

Light-curing one-component varnish for surface sealing of temporary crowns and bridges.

## PROCESSING

### REPAIR AND COMPLETION OF THE TEMPORARY RESTORATION

NOTE: The repair or reinforcement is carried out with **smartrepair®**, and the aesthetic adaptation of the gingival area with **smartrepair® pink**.

#### ► Preparation

Thoroughly rinse and dry the temporary restoration or denture. Roughen and clean the surfaces to be machined; thoroughly remove the grinding dust. Apply **smartbond®** outside the mouth using a smartbrush, forming an opaque layer, and rub it in, then carefully blow with compressed air and allow to dry for approx. 1 minute. When reattaching teeth or manufacturing immediate dentures, sandblast the teeth and provide them with indentations or grooves in order to ensure sufficient mechanical retention to the acrylic.

#### ► Application

Remove the cap of the application syringe and attach the enclosed application needle. This is suitable for pinpoint application and intended for single use. Close the application syringe again after use. For layer thicknesses of more than 2 mm, consider an intermediate polymerization step and apply and cure the material in two or more layers. For this purpose, in order to ensure optimum bonding of the layers do not remove the surface inhibition layer. Wipe the surface with isopropanol only after final completion.

#### ► Curing

Apply **smartrepair®** to the resin surface and light-cure it from all sides with the hand lamp for at least 40 seconds. Then finish the light curing outside the mouth by exposing all sides for 30 sec.

Polymerization of **smartrepair® pink** is induced by light at wavelengths between 360 and 480 nm, and the material can be cured with commercially available equipment within the specified time.

**DENTAL OFFICE:** By means of hand-held lamp per surface approx. 40 sec.

**LAB:** Halogen lamps ≈10 minutes,

UVA tubes approx. 15 – 20 minutes,

stroboscope light ≈3 minutes.

Finish the repair areas and polish the temporary restoration. Alternatively, instead of polishing the surface of the temporary restoration can be sealed with **tempofit® finish**.

### SURFACE SEALING OF THE TEMPORARY RESTORATION

Apply a thin layer of **tempofit® finish** material onto the prepared temporary restoration without air bubbles and then light-cure with the hand lamp for at least 20 seconds.

## SAFETY INFORMATION

- Only for the specified intended use by trained specialists.
- Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- Do not use any adhesion-reducing additives.
- Manual withdrawal of the components is not recommended.
- Close the application syringes again after use.
- Close the bottle again immediately after each use.
- Do not bring individual components into contact with skin or mucous membranes. Irritating to eyes and skin.
- In case of accidental contact, rinse with sufficient running water; seek medical advice if necessary.
- **tempofit® finish** and **smartbond®**: Keep away from sources of ignition
- Refer to the relevant safety data sheet for hazard and safety information.

## NOTES

- Detax shall not be held liable for any damage caused by misuse.
- Read and understand the safety data sheet!

### For users and/or patients:

Any serious incident that has occurred in connection with this product must be reported immediately to [incident@detax.com](mailto:incident@detax.com) and to the competent public authority of the Member State in which the user and/or patient resides.

## CONTRAINDICATION

**smartrepair®** contains (meth)acrylates.

**smartrepair® pink** contains (meth)acrylates and phosphine oxide.

**smartbond® / tempofit® finish** contains (meth)acrylates and phosphine oxide.

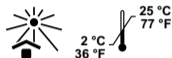
Ingredients may cause allergic reactions in accordingly disposed persons. In such a case, do not use the product any further. 9

**SIDE EFFECTS**


Product may cause allergic reactions.

**Storage:**

smartrepair®/smartrepair®pink and smartbond:

tempofit® finish:

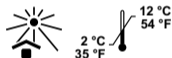**Processing:**

At 23 °C ± 2 °C (73 °F ± 3 °F),  
50 ± 5% rel. humidity

\* from start of mixing at 23 °C ± 2 °C  
(73 °F ± 3 °F),  
50 ± 5% rel. humidity. Higher temperatures shorten, lower temperatures extend the indicated times.

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**detax GmbH**